

# Rovers® Orcellex® Brush

**ENG**

Rovers® Orcellex® Brush, oral cell sampler, sterile, for single use only.  
Do not use if package has been damaged.

**Instructions for use:**


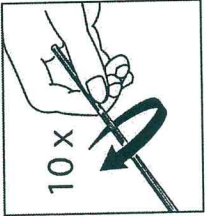
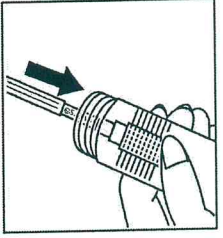
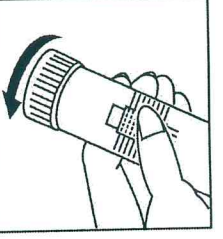

1. Place the Rovers® Orcellex® Brush perpendicular onto the mucosa to be sampled.
2. Press the Rovers® Orcellex® Brush gently into the lining mucosa so that it makes firm contact and displaces the mucosa
3. Rotate the brush through ten complete turns for each site to be sampled. At least five rotations should be used for the floor of mouth and ideally ten if tolerated, maintaining the pressure on the lining mucosa.
4. The Rovers® Orcellex® Brush is for single use only and a new brush and vial must be used for each location examined.
5. Insert the Rovers® Orcellex® Brush head into the vial with preservative fluid and follow the instruction given by the manufacturer of the liquid preparation method.
6. Reseal the vial and discard the blue handle.

FOR SINGLE USE ONLY (DO NOT REUSE)  
Reuse causes contamination and misdiagnosis.

To be used only by healthcare professionals.

Manufactured by:  
Rovers Medical Devices B.V., 5347 KV Oss, The Netherlands

® Registered trademark  
® Patented design

 <p>(1)</p>	 <p>(2)</p>	 <p>(5)</p>	 <p>(6)</p>	 <p>Do not reuse Do not use if package is damaged</p> <p>FOR USE BY HEALTHCARE PROFESSIONALS ONLY</p>	<p>STERILE</p> <p>CE 0044</p>
--	--	--	---	--	-------------------------------